WOHLWEND AG

# ARTISTIC

Technical Manual  
Manual técnico  
Technische Anleitung

*Vision*  
■ 3D ARTISTIC

## Introduction

The Vision-3D Artistic Paint-on Ceramic System consists of different components for refinement and finalisation that are coordinated with each other. This finishing system is ideal for zirconium and lithium disilicate materials. Also conventional metal-ceramics can be finalized with staining technique.

For small space requirements as well as monolithic crowns and bridges, the efficiently and quickly natural effects such as translucency, chroma, halo, mamelons, enamel cracks, opalescence and fluorescence can be achieved easily. The components of the Vision-3D Artistic System consist of fluorescent paints, sculpture masses and a special glaze mass.

The sculpture masses can be applied in a layer thickness of 0.1–0.4 mm over the individually painted surface. Due to their vital colour properties and their stability, three-dimensional color effects from the depths as well as refractive surface structures can be achieved with little effort.

- » Universal application
- » 3D layering effect
- » Increase of the chroma
- » Fluorescent
- » Colour brightening
- » Individual incisal effects
- » Reproduction of natural gingival effects
- » Reliable, reproducible results
- » Simple and economical handling

## Classification

Vision-3D Artistic paint on ceramic is a Class IIa medical device.  
The material is only for professional use in dental laboratories.  
Processing should only be carried out by trained personnel.

## Storage and processing instructions

Please store dry and at room temperature.

- » The pastes must not come into contact with water!
- » Mix pastes and masses well before each use!
- » Do not change Sculpture masses in consistency. If too much fluid is used the modelling stability is no longer guaranteed.
- » For a layer thicknesses over 0.4 mm on zircon or lithium disilicate, Vision-zircon or Vision-LD.Layer ceramic should be used for the respective frame Material!

## Important!

**Mix pastes and compounds well before use.**

Due to their ceramic base, Stains & Shades and Sculpture masses can sediment from the liquid.  
The Vision-3D ARTISTIC stains must not come in to contact with water!  
Always use Vision-3D ARTISTIC Fluid for rewetting!

# Vision-3D Artistic System Components




	Name	Indikation	Translucent	Opacity	Opalescence	Flourescence	Effect	Sculpture
<b>Shades</b>	Shade A	Base /Chroma	✓					
	Shade B	Base /Chroma	✓					
	Shade C	Base /Chroma	✓					
	Shade D	Base /Chroma	✓					
<b>Effect Stains</b>	3D Stain Blue	Incisal / windows	✓				✓	
	3D Stain Sky	Incisal	✓				✓	
	3D Stain White	brightener / Halo		✓	✓	✓	✓	
	3D Stain Gray	Incisal	✓				✓	
	3D Stain Honey	Chroma / Effect	✓				✓	
	3D Stain Redbrown	Effect /Fissure					✓	
	3D Stain Olive	Effect					✓	
	3D Stain Pink	Incisal / Gingiva	✓				✓	
	3D Stain Red	Gingiva	✓				✓	
	3D Stain Violet	Incisal	✓				✓	
	3D Stain Darkbrown	Fissure / Effect					✓	
	3D Mamelon Ivory	Mamelon / Halo			✓	✓	✓	✓
<b>Sculpture Mass</b>	3D Clear	Structure layer	✓		✓	✓		✓
	3D Incisal	Structure layer	✓		✓	✓		✓
	3D Opal	Structure layer	✓		✓	✓		✓
	3D Arctic Blue	Structure layer	✓		✓	✓		✓
<b>Glaze</b>	Glaze Paste				✓			
<b>Stain</b>	Staining Fluid							

## Surface preparation

- » Glazed and smooth surfaces must be lightly roughened or beaded with glass beads, 50 µm and max.2 bar air pressure.
- » Then clean with steamer or distilled water in an ultrasonic cleaner.

## Important!

**Mix stains well before use - store for longer can lead to a slight segregation between the paint powder and the fluid lead!**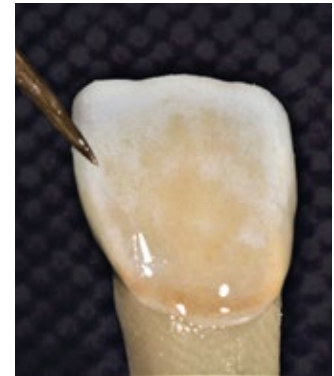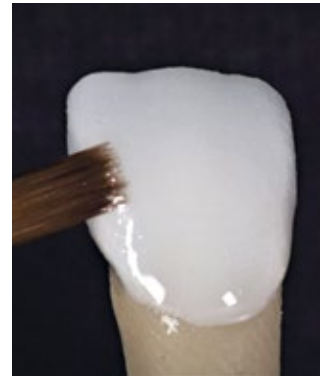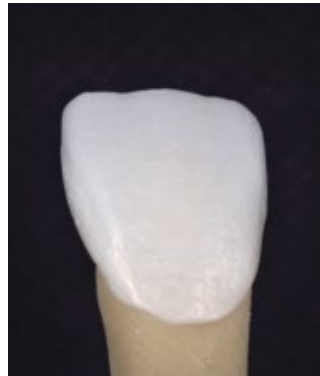

## Characterize / Individualize

- » Wet the surface with a little liquid and glaze.
- » Colouring in the desired colours and effects.
- » Colour fixation firing


Firing Parameter:

Firing	Start Temp	Drying	Heat rate	Final Temp.	Hold Time	Vacuum
Color fixation Zircon	450°C / 840°F	2 Min	40°C / 104°F	800°C / 1472°F	1 Min	yes
Glaze firing Zirconia full anatomical Glasur	450°C / 840°F	2 Min	40°C / 104°F	790°C / 1454°F	1 Min	yes
Color fixation Lithium Disilicate	450°C / 840°F	2 Min	40°C / 104°F	755°C / 1391°F	1 Min	yes
3D Sculptur Mass Firing	400°C / 840°F	4 Min	40°C / 104°F	750°C / 1382°F	1 Min	yes
Glaze firing Lithium Disilicate /Sculptur masses	450°C / 840°F	2 Min	40°C / 104°F	745°C / 1373°F	1 Min	yes

## Important

- » For all commercially available ceramic systems, the firing temperature for Vision-3D ARTISTIC Stains & Glaze should be 10°C / 50°F lower than specified by the manufacturer.
- » On metal ceramic only use Vision-3D ARTISTIC Sculpture masses in small quantities to add contact points or for small corrections together with the glaze firing. Always fire with vacuum. Do not apply in large Quantities!

# Effect color example

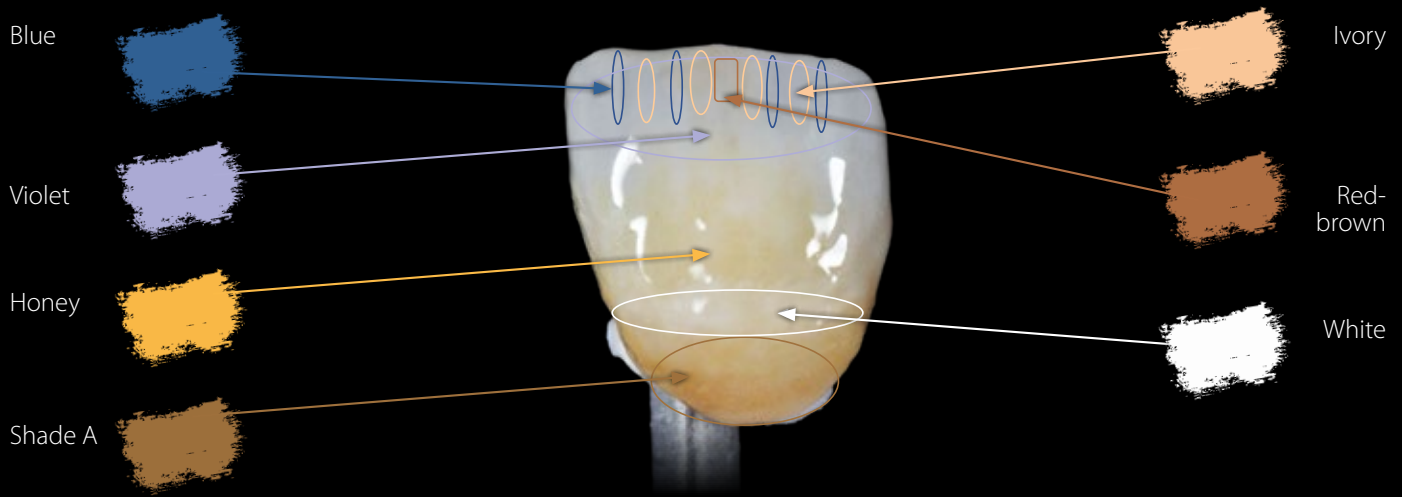## Shades

## Sculptur

## Stains

## Sculpture Mass

By overcoating the ceramic surface which has been individualized with Stains, a three-dimensional colour layer effect is achieved, which can hardly be distinguished to a regular multi-layering veneering. Especially for fully anatomical translucent zirconium as well as lithium disilicate when space is limited, an optically natural aesthetics is easily achieved. The optimum layer thickness of the Sculpture materials are 0.1 to max. 0.4 mm layer thickness. In addition regular layering materials (Vision-LD.Layer or Vision-Zirkon) should be used according to the respective framework material.

The consistency of the sculpture materials should be similar to that of veneering composites.

- » Take out a small amount and knead it with a clean spatula.
- » Apply the desired mass to the surface of the ceramic restoration and modelate the mass with a clean brush into the desired shape.
- » With a dry fanned fine brush you can introduced crosswise and lengthwise structures into the surface.
- » The sculptures masses are fired according to instructions.

## Glaze firing

Depending on the desired degree of gloss, a further glaze firing may be necessary.

See firing table:

Glaze firing parameters:

Firing	Start Temp	Drying	Heat rate	Final Temp.	Hold Time	Vacuum
3D Sculptur Mass Firing	400°C / 752°F	4 Min	40°C / 104°F	750°C / 1382°F	1 Min	yes

Firing	Start Temp	Drying	Heat rate	Final Temp.	Hold Time	Vacuum
Glaze firing Sculptur masses	450°C / 840°F	2 Min	40°C / 104°F	745°C / 1373°F	1 Min	yes

## Gingiva Stains

Gingiva parts made of zirconium or lithium disilicate can now be individualized aesthetically and efficiently with the liquid staining technique. Beauty, naturalness, and vitality can be achieved in simple steps.

With staining and the application of Sculpture effect masses and the use of our special Fluor Glaze nature is copied perfectly. 8 Colours for different pigmentation of the gingiva are available as well as 4 Sculpture Masses to achieve the natural 3D effect

### Vision-3D Artistics Gingiva Shades

Soft Pink

Pink

Rose

Cherry

Rubin

Caribbean

Purple

Brasil

### Vision-3D Artistics Gingiva Sculptur

Magnolia

Pink

Lightred




Darkred

All Vision-3D ARTISTIC colors can be mixed with each other to achieve even more effects.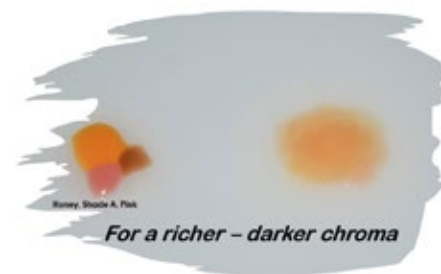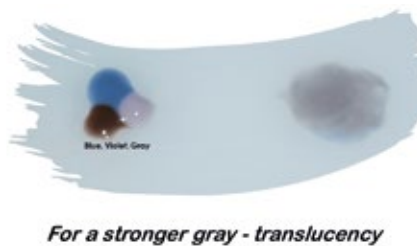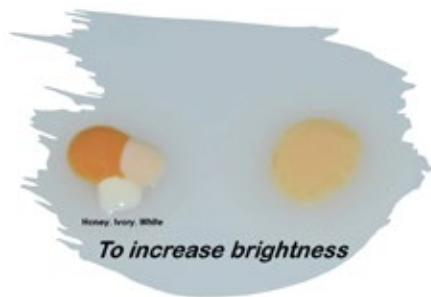

The desired gloss level can also be adjusted by manual polishing.

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# Vision-3D Artistic Systemkomponenten

	Name	Indikation	Translucent	Opacity	Opalescence	Flourescence	Effect	Sculpture
<b>Shades</b>	Shade A	Base /Chroma	✓					
	Shade B	Base /Chroma	✓					
	Shade C	Base /Chroma	✓					
	Shade D	Base /Chroma	✓					
<b>Effect Stains</b>	3D Stain Blue	Incisal / windows	✓				✓	
	3D Stain Sky	Incisal	✓				✓	
	3D Stain White	brightener / Halo		✓	✓	✓	✓	
	3D Stain Gray	Incisal	✓				✓	
	3D Stain Honey	Chroma / Effect	✓				✓	
	3D Stain Redbrown	Effect /Fissure					✓	
	3D Stain Olive	Effect					✓	
	3D Stain Pink	Incisal / Gingiva	✓				✓	
	3D Stain Red	Gingiva	✓				✓	
	3D Stain Violet	Incisal	✓				✓	
	3D Stain Darkbrown	Fissure / Effect					✓	
3D Mamelon Ivory	Mamelon / Halo			✓	✓	✓	✓	
<b>Sculpture Mass</b>	3D Clear	Structure layer	✓		✓	✓		✓
	3D Incisal	Structure layer	✓		✓	✓		✓
	3D Opal	Structure layer	✓		✓	✓		✓
	3D Arctic Blue	Structure layer	✓		✓	✓		✓
<b>Glaze</b>	Glaze Paste					✓		
<b>Stain</b>	Staining Fluid							

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**WOHLWEND** AG
**Wohlwend AG**